

About Biodermis™


For over twenty years, plastic surgeons, dermatologists, and medical professionals worldwide have come to rely on the Biodermis™ family of products as their first line of defense in the management of both new and old keloid and hypertrophic scars.

All scars, regardless of their origin, have the potential to become what physicians refer to as keloid or hypertrophic scars.



ATTENTION PHYSICIANS & PATIENTS

*Epi-Derm™, Xeragel™ and Pro-Sil™
are not available in most pharmacies.
Patients may order from their physician
or direct from Biodermis*

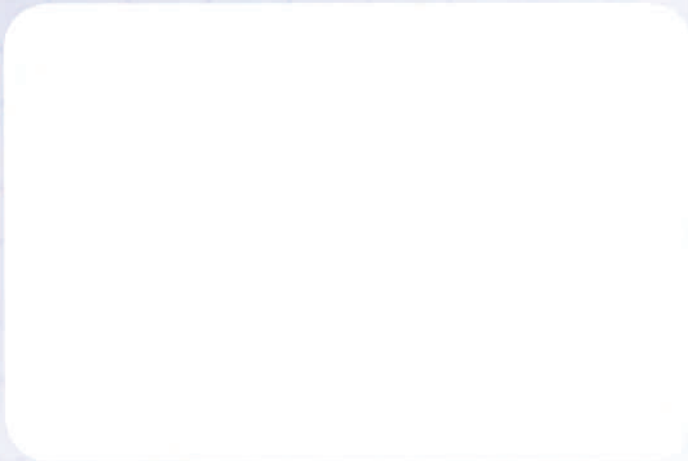

BIODERMIS™
FAMILY OF QUALITY PRODUCTS

For more information or to place
an order within the USA call


1-800-374-3376

www.biodermis.com

email: info@biodermis.com
(702) 260-4466

Problem Scars?

Problem Solved.

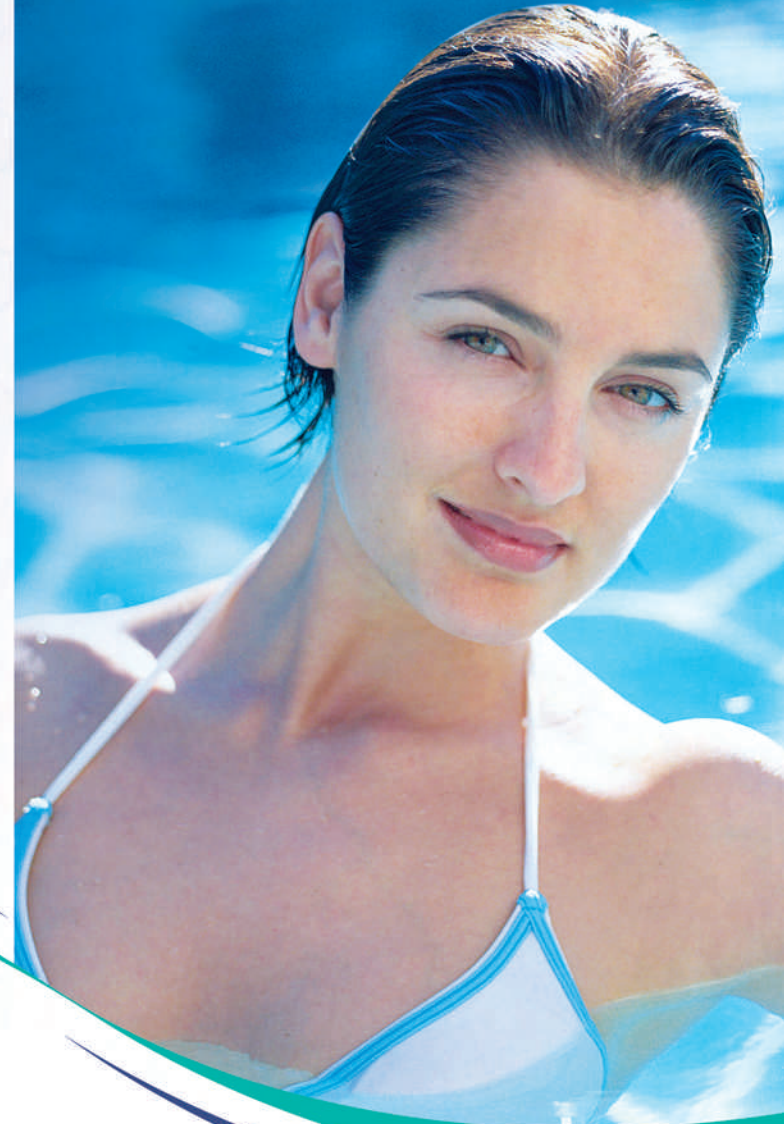


BIODERMIS™
FAMILY OF QUALITY PRODUCTS

Use Texturizer, Patch, and Compressors
and Dermatologists Worldwide

What are Keloid and Hypertrophic Scars?

Keloid and hypertrophic scars are usually thick, raised, unattractive scars. They are frequently associated with a variety of characteristics including hardened, rope-like protrusions, raised surfaces and excessive discoloration.

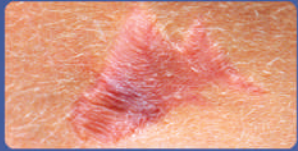

BEFORE



AFTER

No product will ever completely eliminate a scar, however Biodermis™ products are clinically proven to be both safe and effective as a method to manage these unsightly scars, helping them blend in with the surrounding healthy tissue.

The Regimen... The more you wear it the better it works!

Epi-Derm™ Silicone Gel Sheeting, Xeragel™ Ointment or Pro-Sil™ Silicone Gel Stick must be worn a minimum of 12 hours per day for 8-12 weeks. Many patients report continued improvement to the scar beyond the 12-week period. The scar site and gel sheeting material must be washed with SilqueClenz™ prior to each reapplication. If you are using Xeragel™ or Pro-Sil™ the scar site must also be washed prior to application. When the scar stops responding for a continued period of 7-10 days, you have probably reached the point of maximum results and should discontinue using the products. You may safely wear the products beyond the 12-week period as long as you follow the instructions. If you choose to comply with a 24 hour per day regimen you must remove the product, wash it, and wash the scar site at least twice a day, or between each reapplication.

Using Epi-Derm™, Xeragel™ and Pro-Sil™

Many patients choose to use Epi-Derm™ at night for maximum effectiveness, and then use Xeragel™ or Pro-Sil™ when they are in public, since these products are more discreet and easy to reapply. This combination is an ideal way to maximize the effectiveness of

Epi-Derm™ Silicone Gel Sheeting

The products are worn directly over the scar and come in several sheet sizes that can be trimmed as needed to fit your scar. Additionally, we offer a variety of custom shapes and configurations for use with specific types of post-surgical scars.



EPIfoam™

EPIfoam™ offers patients and surgeons an important evolution in post-operative care. As the industry innovator, EPIfoam™

provides an excellent platform over which to apply post-operative compression garments. EPIfoam's™ unique silicone-gel surface provides a gentle adherent patient interface that helps avoid the problems associated with more aggressively adherent foam pads.

Epi-Net™

Epi-Net™ is Biodermis' tubular elastic stretch net designed to serve as a secondary dressing, applying gentle pressure to keep bandages or other medical devices securely in place without adhesive tape. Ideal for securing burn or post-op dressings, hot or cold packs, IV tubes, incontinence pads and decubitus ulcer dressings. Easy to customize, and allows maximum air-flow to the treatment site.



Pro-Sil™ Professional Silicone Therapy for Scars

Pro-Sil™ is a revolutionary, patent-pending solution to scar management which provides the same industry leading silicone gel technology found in Xeragel™, in a convenient, easy to apply slide on applicator.



Xeragel™ Silicone Ointment

This is an ointment version of the same material used in Epi-Derm™ gel sheeting. It is ideal for use on facial

scars and can safely be worn under cosmetics. It is also recommended for hard to cover areas such as the knees, hips, elbows, etc.

Epi-Tape™ & SilqueClenz™

Epi-Derm has its own adhesive that will adhere to most skin types and locations. If your scar is in a location where it requires additional adhesive, Epi-Tape or Epi-Net are the best choices to keep the Epi-Derm Silicone Gel Sheeting in place.



SilqueClenz should always be used to clean the scar site. It will gently and effectively cleanse both the scar site and the Epi-Derm Silicone Gel Sheeting.

