




Promotes the healing process through controlled compression

Fully openable allowing easy examination

Provides optimum comfort and skin-friendliness

# Sophia

SEAMLESS POST-OP BRA

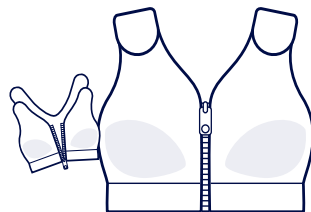

Compression Bra/Surgical Vest

COMPRESSION LEVEL 1 - HIGH



Support/  
compression zones



Breathable





4-way stretch



Seamless



Fully openable

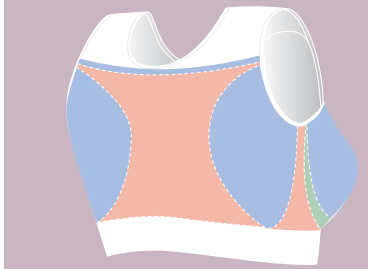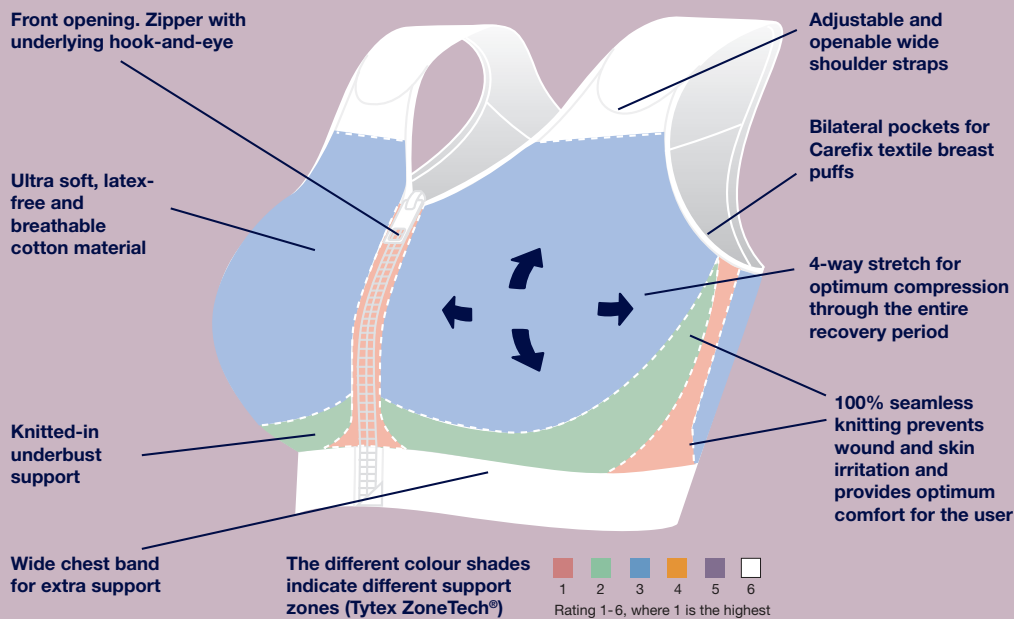

High back for optimum support. Compression zones at the back and sides.



Fully openable at chest and shoulder straps for easy application

The Sophia Post Op Bra/Surgical Vest offers optimum stabilisation and immobilisation of the operated breast. It promotes the healing process by controlled compression on the scar tissue. The bra contributes to wound healing and protection of the surgical site, reducing the risk of edema and post-surgical complications.

With the front opening and openable velcro straps, it is easy to apply the bra for both health care professionals and user, and it allows easy examination of the surgical site.

The 100% seamless knitting increases comfort for the user, prevents pressure points and tissue irritation, resulting in maximum skin-friendliness.

**Indications:**

Medical device for post-surgical use after breast surgery, especially mastectomy. With pockets for Carefix textile breast puffs (not included).

**Precautions:**

If you have any questions regarding use and correct fitting, please consult your doctor or nurse. Make sure that any bandages and/or drainage tubes underneath the bra are placed correctly to avoid pressure points and/or obstructed flow from the drainage tubes. The bra must be removed during radiotherapy, as it contains metal parts. If any unexpected discomfort, wound irritation or similar occurs, please consult your doctor or nurse.

**Recommended sizes**

Band Size	EU	60	65 cm	70 cm	75 cm	80 cm	85 cm	90 cm	95 cm	100 cm	105 cm	110 cm
	US/UK	24 inch	26 inch	28 inch	30 inch	32 inch	33 inch	35 inch	37 inch	39 inch	41 inch	43 inch
Cup Size	AA											
	A			S	M	L	XL	2XL				
	B											
	C											
	D											
E												

\* NOTE: Due to possible post surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size. Important: Always measure yourself to find the right size!

Available colours:



**Composition:**

78% Cotton  
15% Polyamide  
7% Elastane

**Care Handling:**

Coloured:



104°F

White:



140°F

Close zipper and velcro before washing. Wash with similar colours in a washing bag. Use of chlorine-based bleach or chlorine-based disinfection will decrease durability and function.

Dealer: