




Integrated compression zones (Tytex ZoneTech[®]) for the best surgical result

Provides optimum support and skin-friendliness

Bella

SEAMLESS POST-OP BRA

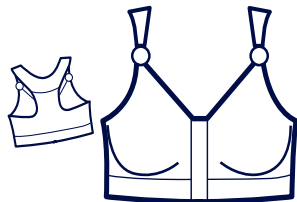

COMPRESSION LEVEL 3 - INTERMEDIATE



Support/
compression zones



Breathable



4-way stretch



Seamless

Front opening with 2 columns hook-and-eye for compression adjustment

Wide shoulder straps for improved support.

Ultra soft, latex-free and breathable microfiber material

4-way stretch for optimum compression through the entire recovery period



Wide chest band for extra support (no hard underwire)

100% seamless knitting prevents wound and skin irritation and provides optimum comfort for the user

Knitted-in under-bust support

The different colour shades indicate different Support/compression zones (Tytex ZoneTech[®])

1 2 3 4 5 6
Rating 1-6, where 1 is the highest

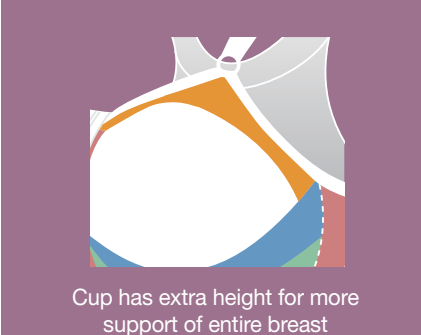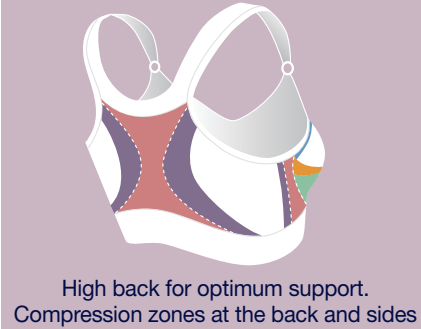

The Bella bra is designed for the time following a surgical procedure to the breast, when the skin is tender and sensitive, and where compression is recommended.

The bra contributes to wound healing and protection of the surgical site, reducing the risk of edema and post-surgical complications.

The 100% seamless knitting increases comfort for the user, prevents pressure points and tissue irritation, resulting in maximum skin-friendliness.

Indications:

Medical device for post-surgical use after breast surgery.

Precautions:

If you have any questions regarding use and correct fitting, please consult your doctor or nurse. Make sure that any bandages and/or drainage tubes underneath the bra are placed correctly to avoid pressure points and/or obstructed flow from the drainage tubes. For breast augmentations, a stabilizer band for breast implants may also be required. Consult your doctor or surgeon before use. Always follow the surgeon's recommendations. The bra must be removed during radiotherapy treatment, as it contains metal parts. If any unexpected discomfort, wound irritation or similar occurs, please consult your doctor or nurse.

Recommended sizes

Band Size	EU	60	65	70 cm	75 cm	80 cm	85 cm	90 cm	95 cm	100 cm	105 cm	110 cm	115 cm
	US/UK	24 inch	26 inch	28 inch	30 inch	32 inch	34 inch	36 inch	38 inch	40 inch	42 inch	44 inch	46 inch
Cup Size	AA												
	A												
	B		XS	S		M		L		XL		2XL	
	C												
	D												
	E												

* NOTE: Due to possible post surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size. Important: Always measure yourself to find the right size!

Available colours:



Composition:

91 % Polyamide
9 % Elastane

Care Handling:



104°F

Close hook and eye before washing. Wash with similar colours in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Dealer: